

Tough-PAC®

Cleaning instructions

Anti-microbial case

The only case system specifically designed for medical use with inherent proven anti -microbial technology to help with infection control - tested to ISO 22196 standard - shown to kill more than 99.6% of contaminating organisms.

General day-to day cleaning

Day-to-day cleaning should still be carried out as part of any infection or decontamination control policy. These recommendations apply to all Tough-PAC® products. For guidelines and procedures, please contact your local infection control team.

- Wear appropriate PPE
- Turn off iPad using the sleep button in the handle aperture
- Clean all external surfaces of Tough-PAC using a moist general cleaning wipe*
- Dry the surfaces and screen using a clean microfibre cloth

* Follow the user instructions of the cleaning product used.

Cleaning for outbreaks or Isolation

- Wear appropriate PPE
- Turn off iPad using the sleep button in the handle aperture
- Clean all external surfaces of Tough-PAC using a moist decontamination wipe, air dry*
- Clean again using a detergent wipe particularly the front screen and case contacts (to remove residues)
- Dry the surfaces and screen with a clean microfibre cloth

* 0.1-0.5 % bleach or a peracetic acid wipe. Follow the user instructions of the cleaning product used.

Cleaning for body fluid contamination

- Wear appropriate PPE
- Turn off iPad using the sleep button in the handle aperture
- Remove all gross contamination*
- Clean all external surfaces of Tough-PAC using a moist chlorinated** cleaning wipe, air dry
- Clean again using a detergent wipe particularly the front screen and case contacts (to remove residues)
- Dry the surfaces and screen with a clean micro-fibre cloth

* If contamination has been rinsed/washed off with a cleaning solution we advise opening the case after cleaning, when completely dry externally to check for any fluid ingress; if present it must be removed. Replace the screen and port plugs on reassembly. Tough-PAC is not 100% waterproof – do not immerse in fluid.

** 0.5% bleach (5200ppm available chlorine) Follow the user instructions for the cleaning product.

Suggested cleaning products

After removing coarse soiling, the surfaces of the Tough-Pac® can be treated by a spray- or wipe-disinfection. All customary cleaning agents can be used for cleaning. In general, the exposure time is as long as necessary, but as short as possible, to make. For cleaning purposes not allowed substances are e.g. benzine, acetone, substances with a chlorine content higher than 10%.

Due to the wide range of substances available on the market and continuous market movements, unfortunately, only an orientation of substances to be used can be given. A binding recommendation is not possible. In this respect, we can not assume any liability for the use and application of all substances.

If you have any questions, please do not hesitate to contact us.

Contact

Baake Medical GmbH & Co. KG

Bacmeisterstraße 3
3312 Lübbecke | Germany

Phone: +49 5741 236027-0

Fax: +49 5741 236027-99

www.baaske-medical.de
info@baaske-medical.de