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Before operating the unit, please read this manual thoroughly and retain it for future reference.

Symbols on the products

Consult the instructions for use

Follow the directions in the instructions for use for parts of the unit on which this symbol appears.

This symbol indicates the manufacturer, and appears next to the manufacturer's name and address.

This symbol indicates the European Community representative, and appears next to the European Community representative's name and address.

This symbol indicates the medical device in the European Community.

This symbol indicates the serial number.

This symbol indicates the version of the accompanying document.

This symbol indicates the date of manufacture.

Storage and transport temperature

This symbol indicates the acceptable temperature range for storage and transport environments.

Storage and transport humidity

This symbol indicates the acceptable humidity range for storage and transport environments.

Storage and transport pressure

This symbol indicates the acceptable atmospheric pressure range for storage and transport environments.

Caution

When you dispose of the unit or accessories, you must obey the laws in the relative area or country and the regulations in the relative hospital regarding environmental pollution.

For the customers in Europe

This product has been manufactured by or on behalf of Sony Corporation, 1-7-1 Konan Minato-ku Tokyo, 108-0075 Japan.

Inquiries to the EU Importer or related to product compliance in Europe should be sent to the manufacturer's authorized representative, Sony Belgium, bijkantoor van Sony Europe B.V., Da Vincilaan 7-D1, 1930 Zaventem, Belgium.

Caution

For cautions and further information on using this product, refer to the Instructions for Use for the monitor.

Caution

Do not use the device in a MR (Magnetic Resonance) environment.

It may cause a malfunction, fire, and unwanted movement.

Precaution

On Safety

- This is the stand for the Sony LCD monitor. For confirming whether your monitor is compatible with this unit, refer to the operating instructions of the monitor.
- When you raise the monitor, raise the lower side of the monitor or the bottom of the stand.
- Clean the surface with a 50 to 70 v/v% concentration of isopropyl alcohol or a 76.9 to 81.4 v/v% concentration of ethanol using a swab method.
- Stubborn stains may be removed with a soft cloth such as a cleaning cloth lightly dampened with mild detergent solution using a swab method, and then cleaning using the above chemical solution.
Never use solvents such as benzene or thinner, or acid, alkaline or abrasive detergent, or a chemical cleaning cloth for cleaning or disinfection, as they will damage the surface.
- Use the supplied screws for mounting. If you use screws other than the supplied ones, the screws may become loose and the device may fall causing injury.

Notes

- Always verify that the unit is operating properly before use. **SONY WILL NOT BE LIABLE FOR DAMAGES OF ANY KIND INCLUDING, BUT NOT LIMITED TO, COMPENSATION OR REIMBURSEMENT ON ACCOUNT OF THE LOSS OF PRESENT OR PROSPECTIVE PROFITS DUE TO FAILURE OF THIS UNIT, EITHER DURING THE WARRANTY PERIOD OR AFTER EXPIRATION OF THE WARRANTY, OR FOR ANY OTHER REASON WHATSOEVER.**
- **SONY WILL NOT BE LIABLE FOR CLAIMS OF ANY KIND MADE BY USERS OF THIS UNIT OR MADE BY THIRD PARTIES.**
- **SONY WILL NOT BE LIABLE FOR THE TERMINATION OR DISCONTINUATION OF ANY SERVICES RELATED TO THIS UNIT THAT MAY RESULT DUE TO CIRCUMSTANCES OF ANY KIND.**


Attaching to the monitor

Accessories supplied


- Base unit (1)
- Neck unit (1)
- M5 × 20 Screws (4)
- M4 × 14 Screws (4)
- Instructions for Use (1)
- Service Contact List (1)

Attaching method


- 1** Assemble the monitor stand.
Insert the base unit to the neck unit **1**, and fix it with the supplied M5 × 20 screws (4) **2**.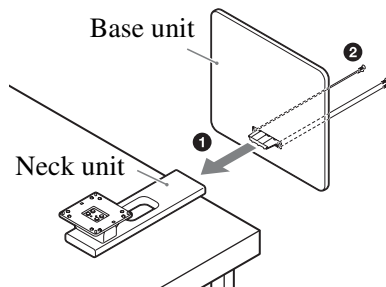

- 2** Place the monitor on a soft cloth, etc., and remove the M4 screws (4) when they are attached to the monitor.
- 3** Attach the monitor stand with the supplied M4 × 14 screws (4).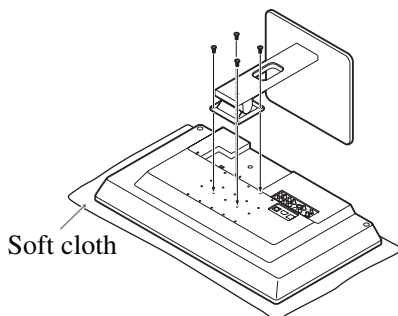

- 4** Connect the cables to the monitor.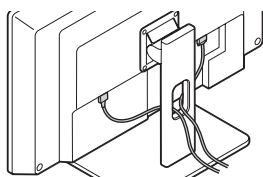


Specifications

Mass	Approx. 2.9 kg (6 lb 6.3 oz)
Storage and transport temperature	-20 °C to +60 °C (-4 °F to +140 °F)
Storage and transport humidity	20% to 90%
Storage and transport pressure	700 hPa to 1,060 hPa


Design and specifications are subject to change without notice.

Dimensions

Unit: mm (Inches)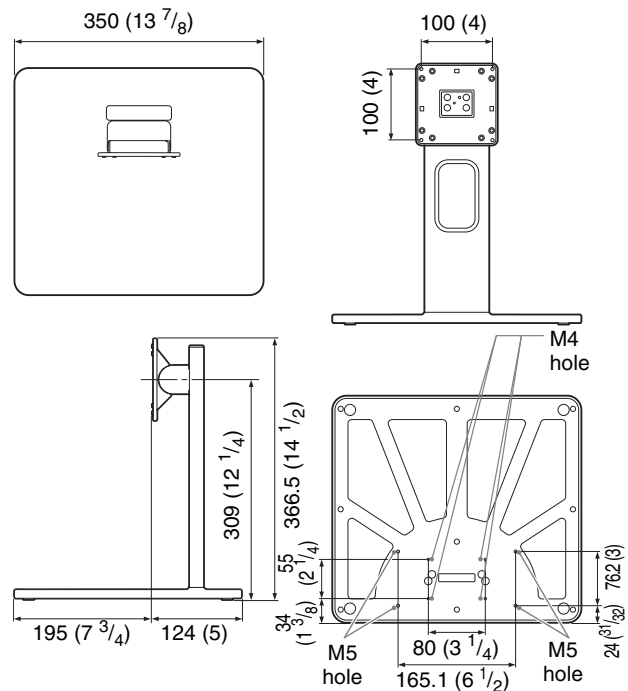

Operating angle adjustment

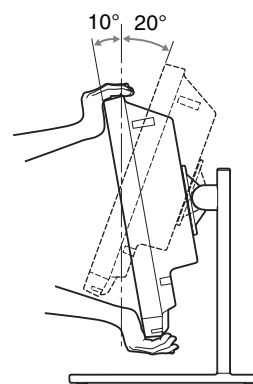You can adjust the angle by holding the top and bottom of the monitor.

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