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Before operating the unit, please read this manual thoroughly and retain it for future reference.

Symbols on the product

Consult the instructions for use

Follow the directions in the instructions for use for parts of the unit on which this symbol appears.

This symbol indicates the manufacturer, and appears next to the manufacturer's name and address.

This symbol indicates the EU Importer, and appears next to the EU Importer's name and address.

This symbol indicates the European Community representative, and appears next to the European Community representative's name and address.

This symbol indicates the date of manufacture.

This symbol indicates the serial number.

This symbol indicates the version of the accompanying document.

Storage and transport temperature

This symbol indicates the acceptable temperature range for storage and transport environments.

Storage and transport humidity

This symbol indicates the acceptable humidity range for storage and transport environments.

Storage and transport pressure

This symbol indicates the acceptable atmospheric pressure range for storage and transport environments.

Caution

When you dispose of the unit, you must obey the laws in the relative area or country and the regulations in the relative hospital regarding environmental pollution.

Warning

When not using the remote for an extended period of time, remove the batteries to avoid possible damage from battery leakage and corrosion.

Caution for batteries

Use only alkaline and zinc carbon batteries. Danger of explosion if battery is incorrectly replaced.

Replace only with the same or equivalent type recommended by the manufacturer.

When you dispose of the battery, you must obey the law in the relative area or country.

Caution

Do not use the device in a MR (Magnetic Resonance) environment.

It may cause a malfunction, fire, and unwanted movement.

Usage Precautions

If the unit is suddenly taken from a cold to a warm location, or if ambient temperature suddenly rises, moisture may form on the outer surface of the unit and/or inside of the unit. This is known as condensation. If condensation occurs, remove the batteries and wait until the condensation clears before operating the unit. Operating the unit while condensation is present may damage the unit.

If this unit is used together with an electrosurgical knife, etc., the picture may be disturbed, warped or otherwise abnormal as a result of strong radio waves or voltages from the device. This is not a malfunction.

When you use this unit simultaneously with a device from which strong radio waves or voltages are emitted, confirm the effect of this before using such devices, and install this unit in a way that minimizes the effect of radio wave interference.

Care of the unit


If the surface of the unit is dirty

- Clean the surface of the unit with isopropyl alcohol at a concentration of 50 to 70 v/v% or ethanol at a concentration of 76.9 to 81.4 v/v%.
- Clean off dirt on the exterior using a soft, cleaning cloth lightly moistened with diluted neutral detergent, then clean using the above chemical agent.
- Do not wipe the surface of the unit with benzine or thinners, acidic cleaning solution, alkaline cleaning solution, cleaning agent with abrasive agent, chemical cloth etc, as these will damage the surface.
- Do not wipe vigorously while any dust adheres to the cloth. This may scratch the surface of the unit.

Overview


The CCMA-2DAR 2D Camera Adapter is an optional accessory used when an MCC-1000MD HD Video Camera is attached to a microscope. The unit attaches to the microscope to enable shooting of objects observed under the microscope by the camera.

System Configuration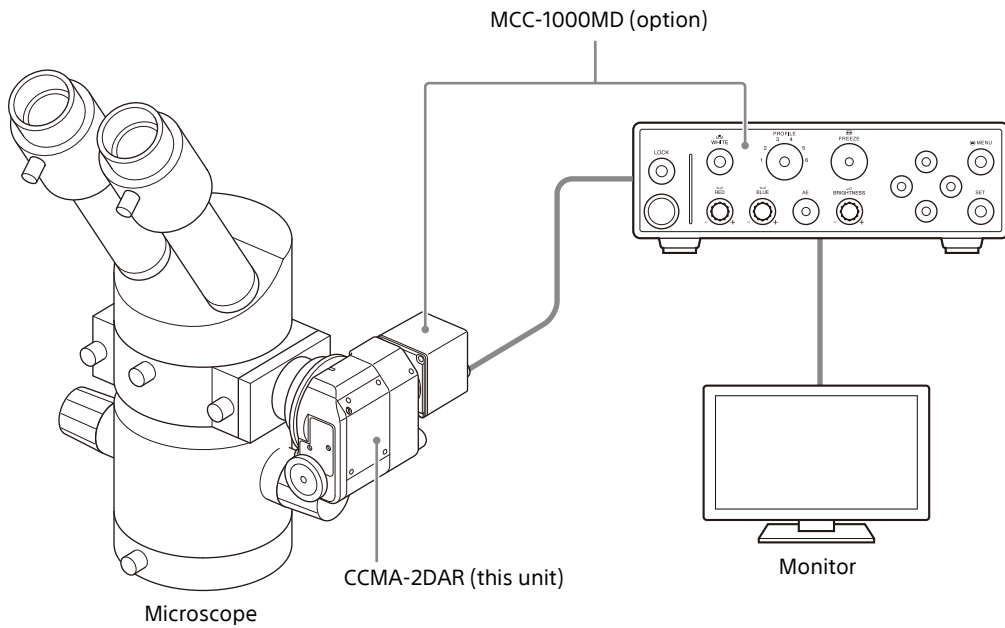


Location and Function of Parts

Main unit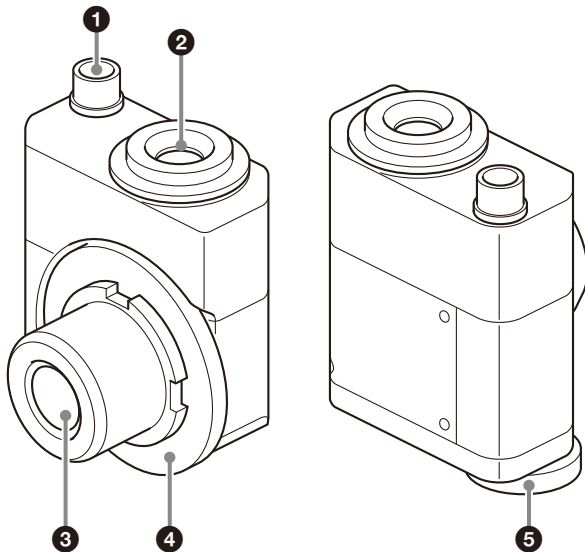

- 1 DC input Controller connector (round type, 5-pin)**
Connect to the supplied controller.
- 2 Camera connector (C mount)**
Connect to the MCC-1000MD camera head.
- 3 Microscope connector**
Connect to a microscope. If the microscope is an M38 type, use the supplied M38 adapter attachment.
- 4 Iris adjustment ring**
Adjusts the iris of the camera.
- 5 Focus adjustment knob**
Adjusts the focus of the camera.


Controller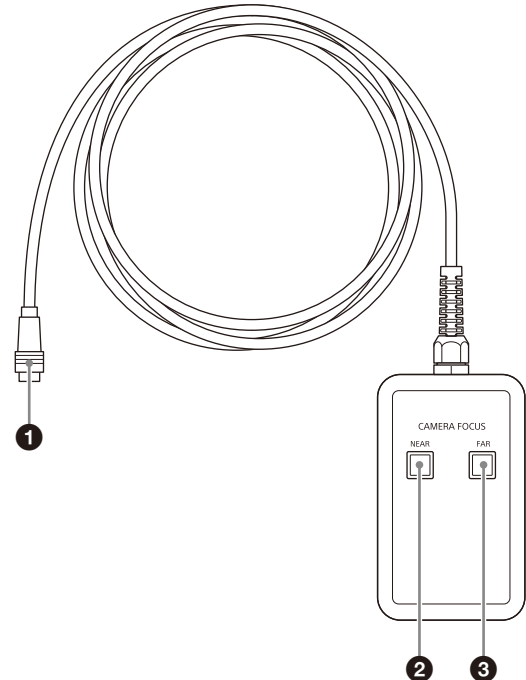

- 1 Adapter connector (round type, 5-pin)**
Connect to the camera adapter.
- 2 NEAR button**
Moves the focus position to the NEAR side while the button is pressed.
- 3 FAR button**
Moves the focus position to the FAR side while the button is pressed.

Inserting the batteries

Open the cover on the rear side of the controller, and insert two size AA (R6, LR6) batteries (not supplied) with the + and - terminals aligned with the ⊕ and ⊖ marks inside the battery compartment.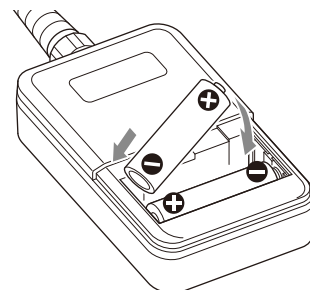

Note


When replacing the batteries, always change both at the same time. Do not use an old battery and a new battery, and do not use different types of battery, such as an alkaline battery and a manganese battery.

Attachment and Adjustment

Before attaching, remove the caps from the camera connector and microscope connector.

Attaching the camera head


Align the lens mount thread of the camera head with the camera connector of the unit, then turn the camera head clockwise to secure it to the unit.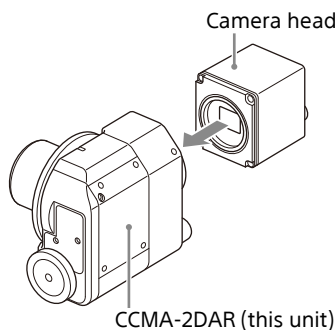


Attaching to a microscope

1 Align the protrusion on the beam splitter of the microscope with the cutout in the microscope connector of the unit, and insert.

If the microscope is an M38 type, attach the supplied M38 adapter attachment beforehand.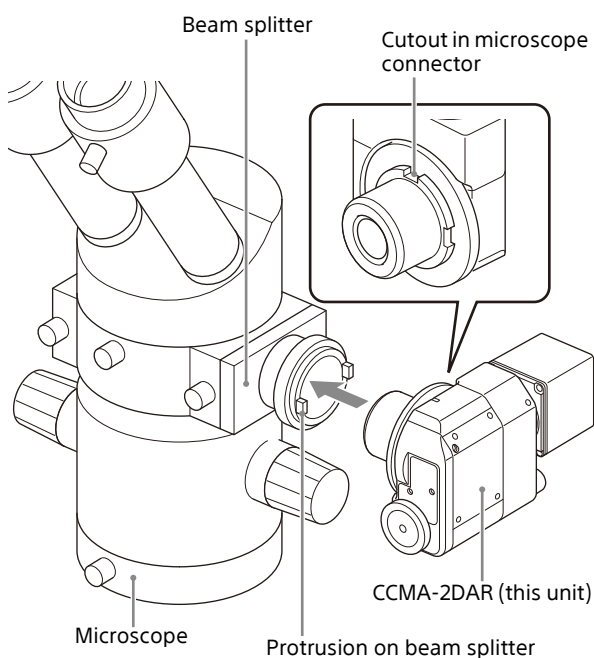

2 Turn the lock ring attached to the beam splitter of the microscope to secure the unit to the microscope.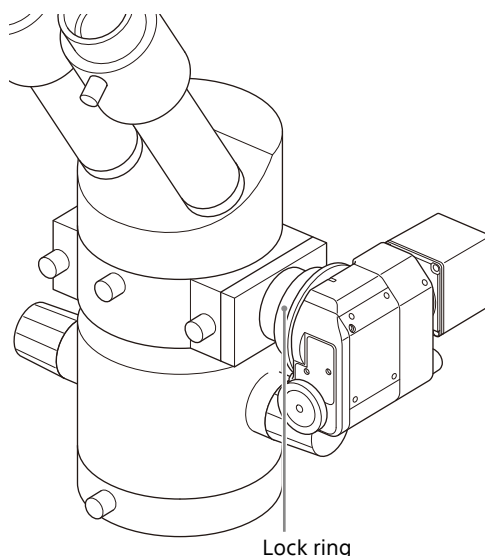

Attaching the controller

Insert the adapter connector of the controller into the controller connector of the unit.

Adjusting the camera image

Adjust the orientation and position of the camera image after attaching the camera head to the unit.


Use a 1.5 mm hex wrench to make adjustments.

Adjusting rotation

Adjust the rotation if the orientation of the image on a monitor does not match the image in the eyepiece of the microscope.

- 1** Loosen the C mount lock screw of the unit using a hex wrench.
The camera head attached to the unit can then be rotated.
- 2** Rotate the camera head while monitoring the monitor image until the orientation of the images match.

- 3** Tighten the C mount lock screw in the position where the orientation of the images match to secure the camera head.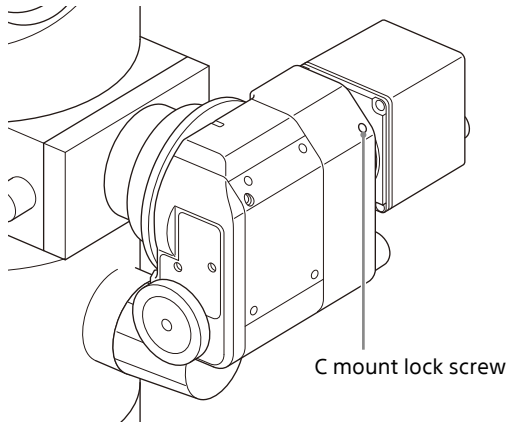


Centering the monitor image

If the position of the image on a monitor is offset, adjust the position using the X and Y-axis adjustment screws.

If the image is offset horizontally, adjust the X-axis adjustment screw using a hex wrench.

If the image is offset vertically, adjust the Y-axis adjustment screw using a hex wrench.

X-axis adjustment screw

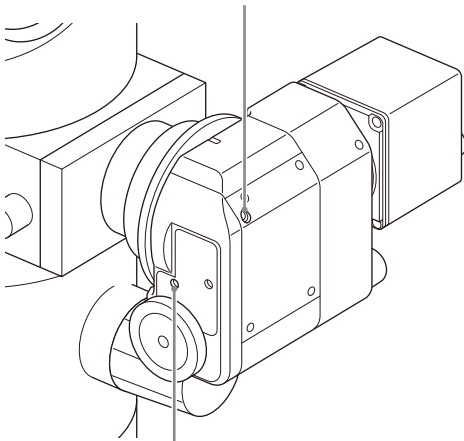Y-axis adjustment screw

Operation

Adjusting the focus

Using the controller

Adjust the focus using the NEAR button and FAR button of the controller connected to the unit. The focus position moves within the drive range in the corresponding direction while the NEAR button or FAR button is pressed.

Using the focus adjustment knob

Turn the focus adjustment knob of the unit to adjust the focus.

Adjusting the iris

Turn the iris adjustment ring of the unit to adjust the iris.

You can set the adjustment to F2.8, F4, F5.6, or F8.

Troubleshooting

If focus cannot be adjusted using the controller

Check the following.

- Batteries are inserted in the controller
- Batteries are inserted in the correct orientation
- Specified batteries are being used
- Cable between the controller and the unit is securely connected

If color shading is a concern

Adjust the shading of the optical system manually.

For details about adjustment, refer to the MCC-1000MD Instructions for Use.

Specifications

General

Power requirements

3 V DC (two size AA (R6, LR6) batteries)

Power consumption

0.03 W (Motor: No load current × rated voltage)

Operating conditions

Temperature 0 °C to 40 °C (32 °F to 104 °F)

Humidity 20% to 80% (no condensation allowed)

Pressure 700 hPa to 1,060 hPa

Storage and transport conditions

Temperature -20 °C to +60 °C

(-4 °F to +140 °F)

Humidity 20% to 80% (no condensation allowed)

Pressure 700 hPa to 1,060 hPa

External dimensions

Main unit: 70 mm × 34 mm × 74.5 mm

(2 ⁷/₈ in × 1 ³/₈ in × 3 in)

(excluding protrusions)

Controller: 69 mm × 28 mm × 115 mm

(2 ³/₄ in × 1 ¹/₈ in × 4 ⁵/₈ in)

(excluding cable)

Cable: ø5.9 mm × 4 m (ø¹/₄ in × 13.1 ft)

Mass

Main unit: 430 g (15.2 oz)

Controller: 250 g (8.8 oz) (excluding batteries)

Lens mount

C mount

Focal length

40 mm

Aperture opening (F-stops)

2.8 to 8

Supplied accessories

M38 adapter attachment (1)

Camera connector cap (1)

Microscope connector cap (1)

Instructions for Use (1)

Warranty (1)

Service Contact List (1)

Information for Customers in Europe (1)

Design and specifications are subject to change without notice.

Notes

- Always verify that the unit is operating properly before use. SONY WILL NOT BE LIABLE FOR DAMAGES OF ANY KIND INCLUDING, BUT NOT LIMITED TO, COMPENSATION OR REIMBURSEMENT ON ACCOUNT OF THE LOSS OF PRESENT OR PROSPECTIVE PROFITS DUE TO FAILURE OF THIS UNIT, EITHER DURING THE WARRANTY PERIOD OR AFTER EXPIRATION OF THE WARRANTY, OR FOR ANY OTHER REASON WHATSOEVER.
- SONY WILL NOT BE LIABLE FOR CLAIMS OF ANY KIND MADE BY USERS OF THIS UNIT OR MADE BY THIRD PARTIES.
- SONY WILL NOT BE LIABLE FOR THE TERMINATION OR DISCONTINUATION OF ANY SERVICES RELATED TO THIS UNIT THAT MAY RESULT DUE TO CIRCUMSTANCES OF ANY KIND.

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© 2019 Sony Corporation