


Medical Built In Isolator MI 1005 E

User Manual

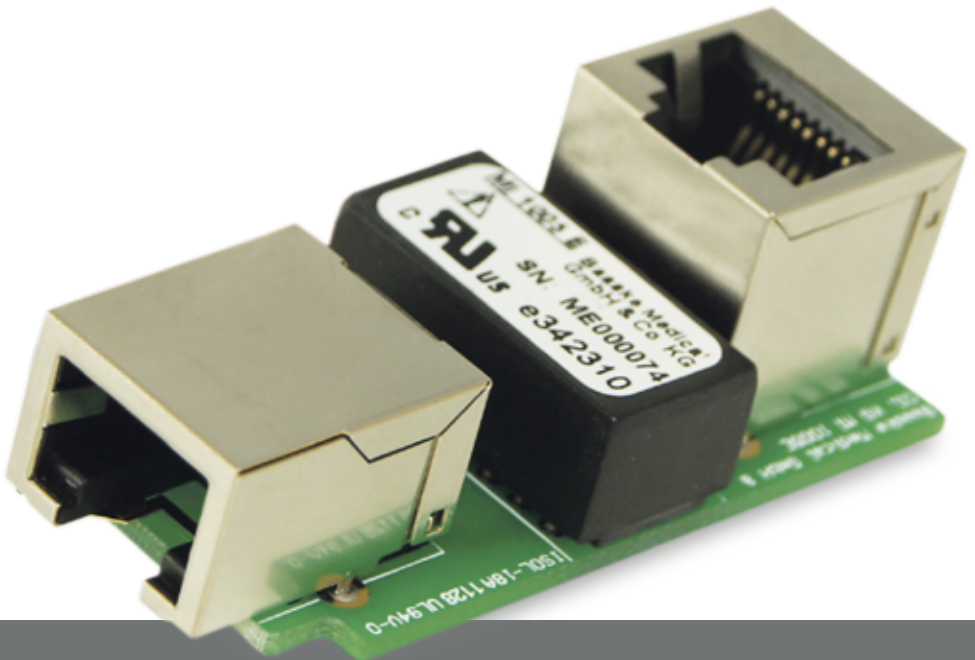



<< **Read carefully prior to initial operation start-up!** >>

// 1. Intended Use

The MI 1005 E is an appliance especially developed for use in medical technology. The purpose of the MI 1005 is to isolate the Ethernet signal, in compliance with the relevant norms for electrical safety a medical appliance or system equipped with a network interface to a non-medical appliance or system. The MI 1005 E shall be used as a built in device only using standard Harting assembly boxes.

// 2. Installation

The MI 1005 E Medical Built In Ethernet Isolator can be mounted into a panel cutout using following standard Harting assembly boxes:

- ▶ Harting Assembly Box Compact (Art.No.: 2006634)
- ◀ Harting Assembly Box Easy Install (Art.No.: 2006635)

The details for panel and case mounting can be obtained from the Harting Assembly box installation instructions.

// 3. Maintenance

The MI 1005 E Medical Built In Ethernet Isolator is maintenance-free, when used as intended. For any questions, please contact your supplier.

// 4. Environmental Protection

For disposal of the MI 1005 E stipulations subject to public law may involve special instructions. Before the final de-commissioning of the appliance, contact the manufacturer.

// 5. Safety instructions

The MI 1005 E is not protected against liquid intrusion. The MI 1005 E may only be used in network environments as specified in the Technical Data listed below.

// 6. Manufacturers Contact

Baaske Medical GmbH & Co. KG

Bacmeisterstraße 3
32312 Lübbecke

Tel.: +49 5741 236027-0

Fax: +49 5741 236027-99

sales@baaske.net

www.baaske-medical.de

// 6. Technical Data:

Manufacturer:	Baaske Medical GmbH & Co.KG Lübbecke, Germany
Type:	MI 1005 E
Operating Mode:	Continuous
Operating Temperature:	-10°C to +85°C
Storage Temperature:	-25°C to +85°C
Operating Air Pressure:	700 hPa to 1060 hPa
Storage Air Pressure:	500 hPa to 1060 hPa
Operating Air Humidity:	10% to 90%
Storage Air Humidity:	10% to 95% (not condensing)
Dimensions (L x W x H):	51 x 23.4 x 18.2 mm
Weight:	20 g
Voltage strength between the network connections:	5 kV AC (50/60 Hz 1 min.)
Connection sockets:	2 x RJ45
Material:	UL94V-0 / RoHS compliant
Network specifications:	IEEE 803.2ab 10/100/1000-BaseT, Twisted-Pair, auto-conf
MTBF:	0,21 x 10 ⁸ (1 error in 21 000 000 hrs)
Product rating:	Passive ethernet isolating device, insulation rating DI (250 V AC / 300 V DC) tested at 5 kVAC
Marking:	CE, UL (E342310), RoHS compliant

Meaning of used symbols



Accompanying documents



© 2014 Baaske Medical GmbH & Co.KG. All rights reserved.

Information in this document has been carefully checked for accuracy; however, no guarantee is given to the correctness of the contents. Photographs and schematic diagrams used in this document are for illustration purposes only. The actual product may vary. This document contains proprietary information protected by copyright. No part of this manual may be reproduced by any mechanical, electronic, or other means, in any form, without prior written permission of Baaske Medical.

This document is subject to change without notice.

Rev. 2 07_2014



Baaske Medical GmbH & Co. KG

Bacmeisterstraße 3
32312 Lübbecke

Tel: +49 5741 236027- 0

Fax: +49 5741 236027- 99

sales@baaske.net

www.baaske-medical.de