MEDICAL COMPUTER HARDWARE

# MediTouch

Handbuch / Manual

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*© 2017 Baaske Medical GmbH & Co.KG. All rights reserved.*

*Information in this document has been carefully checked for accuracy; however, no guarantee is given to the correctness of the contents. Photographs and schematic diagrams used in this document are for illustration purposes only. The actual product may vary. This document contains proprietary information protected by copyright. No part of this manual may be reproduced by any mechanical, electronic, or other means, in any form, without prior written permission of Baaske Medical.*

*This document is subject to change without notice.*

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**Note:**

*This symbol cautions the user that important information regarding the operation and / or maintenance of this equipment has been included. Information preceded by this symbol should be read carefully to avoid damage to the equipment.*

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**Safety Compliance:**

*This product meets the requirements of EN60601-1-2.*

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**Recycling:**

*Follow local governing ordinances and recycling plans regarding the recycling or disposal of this equipment.*

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---

## Content

|                                       |    |
|---------------------------------------|----|
| Intended Use .....                    | 9  |
| Safety instructions .....             | 9  |
| Initial operation .....               | 9  |
| Cleaning / Maintenance and Care ..... | 10 |
| Liability and Warranty .....          | 11 |
| Disposal .....                        | 11 |
| Notes on conformity .....             | 11 |
| Contact .....                         | 12 |

## Intended Use

This device has been designed to be used with a computer/laptop/mobile device within the patient environment. Please note that we shall not be liable in cases of not intended use:

- Do not modify the device without our consent and do not use any auxiliary equipment we have not approved or supplied.
- Only use spare parts and accessories we have supplied or approved.
- Follow all the instructions in these operating instructions, especially the safety information. Any other use is deemed improper and can cause personal injury or damage to the product.
- Do not use the device in extreme environmental conditions.

**Attention!** BLT02 type keyboards are not suitable for use in operating rooms.

## Safety instructions

- After transporting the device, wait until it has reached room temperature before switching it on.
- Protect the cable from being stepped on or tripped over.
- To avoid damaging the cables, do not place any objects on them.
- Never open the housing. This invalidates the warranty and will invariably render the keyboard unusable.

## Initial operation

Before using the product, make sure that it is intact. The Meditouch products are powered by the USB port of your PC and can be put into operation directly via plug and play.

Wireless models are covered by an exception. Before using your wireless product, two AAA functional batteries have to be inserted into the designated battery compartment on the back. Plug the supplied dongle into one of the USB interfaces on your PC and wait a moment while the product is connecting to your PC.

## Cleaning / Maintenance and Care

In order to assure the preferred low activation force, the thickness of the silicone rubber material at the foot of the keys is very thin. This area can easily be injured or damaged with a sharp object. This would lead to a damage of the entire keyboard, so please be aware of that specific character of silicone rubber keypads and keep sharp or spiky objects away from the rubber. In a second process the device should be disinfected. This can be done best with a spray- or wipe-disinfection. Since we have taken care specifically for the cleaning and disinfection qualities of our products already in the design process, we recommend cleaning and disinfecting by wiping. Our specific low profile key design allows and supports that method especially. It makes that process even more efficient and can be executed frequently to provide permanent hygienic conditions on your keyboard/mouse surface.

In case immersing is time by time necessary or preferred, the time of dipping the product in the disinfecting liquid should be as long as necessary but as short as possible. It should be rinsed by clear water immediately and wiped with a dry cloth.

For cleaning purposes not allowed substances are e.g. benzine, acetone, substances with a chlorine content higher than 10%.

Due to the wide range of substances available on the market and continuous market movements, unfortunately, only an orientation of substances to be used can be given. A binding recommendation is not possible. In this respect, we can not assume any liability for the use and application of all substances.

If you have any questions, please do not hesitate to contact us.

**Contact:**

Mon to Fri, 8am - 5pm

[sales@baaske-medical.de](mailto:sales@baaske-medical.de)

Phone: +49(0)5741 236027 0

## Liability and Warranty

You are only entitled to a voluntary warranty service if your unit has been operated in accordance with the operating instructions in the manual and the valid legal requirements. Please note our general terms and conditions: [www.baaske-medical.de/general-terms-and-conditions](http://www.baaske-medical.de/general-terms-and-conditions)

In the warrenty case, please contact:

### Support

[support@baaske-medical.de](mailto:support@baaske-medical.de)

### WARNING!

Never try to open or repair the device yourself. There is a risk of electric shock!

## Disposal

At the end of its life, under no circumstances dispose of the device with your ordinary household rubbish. Consult your local authority to find out an environmentally sound disposal for electrical and electronic scrap.

## Notes on conformity

Your device fulfils the requirements for electromagnetic compatibility.

Full Declarations of Conformity are available on request.

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Baaske Medical GmbH & Co. KG